


EP00602 / D

LEGREST BZ8 / INOVYS2 35				Plan No.
				EP00602 / D
Scale:	1:3	Drafted	Verified	Confirmed
Date	05/08/2021	05/12/2022	05/12/2022	
Initials	AS	AZ	AS	

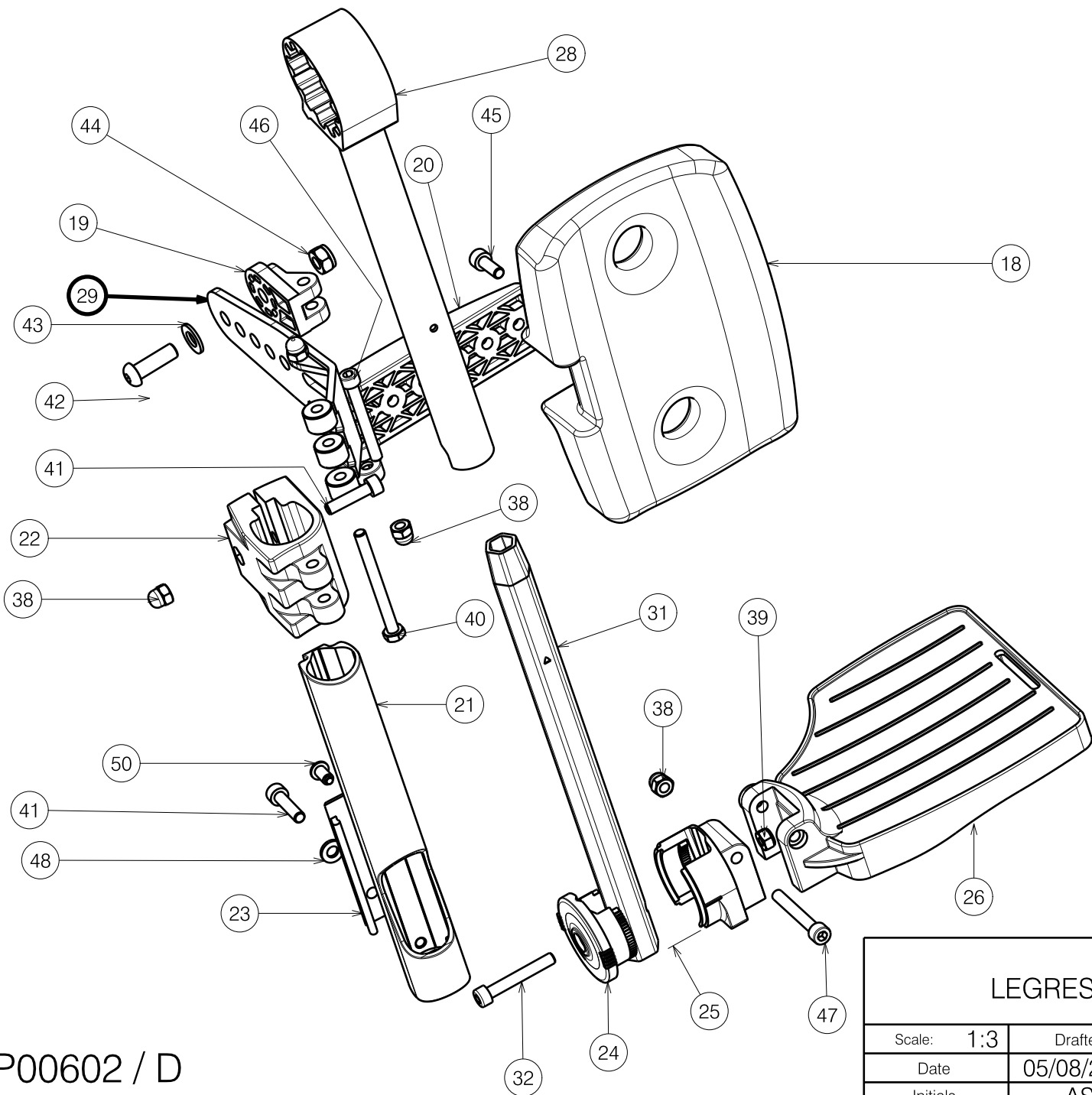



EP00602 / D


LEGREST BZ8 / INOVYS2 35				Plan No.
				EP00602 / D
Scale: 1:3	Drafted	Verified	Confirmed	2/3
Date	05/08/2021	05/12/2022	05/12/2022	
Initials	AS	AZ	AS	



Material			
No.	Plan	Article	Quantity
1	X004665	1905744	1
2	X004710(R)	1905738	1
	X004701(L)	1905737	1
3	X004692	1906013	1
4	X004773	1905885	1
5	X004417	1719200	1
6	X004660	1905195	1
7	X004423	1717300	1
8	X005055	1917169	1
9	X005028	G103900	1
10	X005047	1908420	1
11	X004795	1905970	2
12	X005880	1915832	2
13	X004423	1717300	1
14	X004690	1905735	1
15	X005046	G103800	1
16	X004713	1905734	1
17	X004742	1739500	2
18	X004785	1905949	1
19	X004787	1905947	1
20	X004784	1905950	1
21	X004666	1905731	1
22	X004739	1905739	1
23	X004658	1904968	1
24	X004464	1904212	1
25	X004465	1904213	1
26	X004571(R)	1904647	1
	X004570(L)	1904646	1
27	X005025	1772000	1
28	YP09153	2636654	1
29	P00044872	2659754	1
30	Y018552(R)	1771554	1
	Y018551(L)	1771654	1

Material			
No.	Plan	Article	Quantity
31	YP06404	2730199	1
32	M6x40	1003909	2
33	M6RT	1905500	2
34	M6x25 KT	1905501	2
35	M5x10	1999977	2
36	M6x16	1994752	2
37	M5	1001679	1
38	M6	1008492	3
39	M6	1001680	1
40	M6x65 DIN931	1991860	1
41	M6x25 DIN912	1001494	2
42	M8x25	1990501	1
43	M8 DIN125	1001554	1
44	M8	1001682	1
45	M6x16 DIN912	1001492	2
46	M6x50	1007970	1
47	M6x45	1003109	1
48	M6	1001564	1
49	D4x24	G0409	1
50	X004687	1905639	1

EP00602 / D

LEGREST BZ8 / INOVYS2 35				Plan No.
3/3				EP00602 / D
Scale: 1:3	Drafted	Verified	Confirmed	
Date	05/08/2021	05/12/2022	05/12/2022	
Initials	AS	AZ	AS	