

# INSTRUCTION MANUAL

## Side table 377

First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the side table, read the instruction manual carefully: it will familiarise you with this product. You will also be given hints about the care and service of your side table. Please remember that observance of our hints would keep your side table in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

### Technical details

Height:	65 - 94 cm
Length:	81 cm
Width:	43 cm
Folded:	57 x 85 cm
Table:	62 x 41 cm
Max load:	<b>max. 10 kg</b>

### Contents of consignment

- Base-frame (including 4 castors)
- Upper frame (incl. adjustable tray)
- Instruction manual

### Purpose

This side table can be pushed in beneath a patient's bed and used as a tray for holding various objects. It cannot be used for supporting the patient or other persons. It can also not be used for conveying persons or objects. The manufacturer does not accept liability for damage caused by improper handling.

→ **N.B: The wing nuts must be tightened before any objects can be placed on the trays.**

→ **N.B:** Check that the inserted tubes are not jammed or skew and that they are pushed in fully.

→ **N.B:** Make sure that the wing nuts are tightened properly before anything is placed on the trays.

→ **The stability of the product is affected when the tray is adjusted (danger of tipping over).**

→ The tray should be tilted only so far that nothing will fall off.



### Safety

→ Do not stand on the tubes of the base-frame, since the table could tip over -- danger of injury !

→ Do not lean against or on the trays -- danger of tilting !

→ Do not lift the table by means of the trays. These parts are adjustable and might cause injury or damage when raised up if they are not properly secured.

→ Do not place any objects larger than the trays on them. These could fall and cause injury / damage.

→ Do not exceed the bearing capacity of the table (10 kg) with heavy objects -- danger of tipping over !

→ Do not convey persons or objects -- danger of injury !

→ The side table must only be set up indoors on level surfaces. When in use, check that all 4 castors press evenly on the floor.

→ Repair work may only be undertaken by the authorised dealer.

→ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

### Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your side table is preserved in a perfectly functional condition.

### Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions

of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: [www.rki.de](http://www.rki.de)). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

### Repairs / Service / Inspection

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Check the castors (stability, tracking, fixing screws)
- Setting of the wing nuts (Condition, thread, holding security)
- Frames, (deformation, stability)
- Trays, (closed surfaces, splinters, tears)
- Damaged varnish (remove possible rust and repaint)

### Storage

**between + 5° and + 45° Celsius  
20% - 75% relative humidity**



### MANUFACTURER:

N.V. VERMEIREN  
Vermeirenplein 1 / 15  
B-2920 Kalmthout (Belgium)  
Tel.: +32 (0) 3/62020 Fax: +32 (0)3 6664894

### MARKETING:

N.V. VERMEIREN  
Vermeirenplein 1 / 15  
B-2920 Kalmthout (Belgium)  
Tel.: +32 (0) 3/6202020 Fax: +32 (0)3 6664894